


Thank you for choosing INSPECTIS®

This product has been tested with Windows 7/10 Professional. Below you will find step by step instructions on how to install.

A. Installation of digital microscope device drivers

1. Double click on “*drv\Installer.exe*” under Capillaroscope Device Drivers 2.8.5 folder and follow up the on-screen instructions.
2. Connect the Capillaroscope into a free USB3.0 or USB3.1 port of your PC when it is asked from the installer software.
3. Once the installation is complete, it is recommended to restart your pc.

B. Installation of application software INSPECTIS

1. Install the software by running “**Inspectis x.x Setup**” installation file.
2. Once installation is finished, right click on INSPECTIS icon  on your computer desktop and select **Run as Administrator** first time.

On “Select Inspectis Version” dialog box, select:

- a. “**INSPECTIS CAP Pro**” if you have ordered the capillaroscopy **Pro** license. Insert the provided software protection USB dongle into a free USB port of your computer. Enter license number and other requested information. Follow up on-screen instructions to complete installation.
 - c. “**INSPECTIS CAP Basics**” if you have ordered a capillaroscopy **Basics** license. Enter license number and other requested information. Follow up on-screen instructions to complete installation.
 - d. 15days trial. All software functions will be enabled for 15 days. License number is not required.
3. Software will be registered and activated automatically over internet once you provide license number and other requested information. If on-line registrations is prohibited or not possible, complete the Off-line Registration form and email it to info@inspect-is-com to get an activation key.

For more information refer to instruction’s manual of **INSPECTIS®** software.

If you have problems running on other versions, please contact your dealer or send an email to info@inspect-is-com