

## Release Notes for INSPECTIS CAP Pro Version 6.0 through 6.2.16

### New:

1. Export patient list to Excel directly from patient manager.
2. An uncalibrated image can be calibrated by software using a known reference distance on the image.
3. Number of displayed decimals for measurements (accuracy) can be set from 1 to 6 (0 to 0.00001) under Preferences. Default is set to 2 decimals (0.01).
4. Grid overlay is moveable across image field. Center of the Grid pattern is highlighted by a circle.
5. Current position of Grid overlay is saved when program closed.
6. Grid overlay can be centered automatically by mouse right-click.
7. Rectangle overlay can be centered automatically by mouse right-click.
8. Current position of Rectangle overlay is saved when program closed.

### Improvements:

1. Image gallery with an option to choose examination date.
2. User interface for "Select Working Folder" is improved.
3. Size and position of the icons on main window is adjusted.
4. License Agreement Updated.
5. In Preferences "Additional Information for New Case" is renamed to "Additional information for Patient".
6. In Preferences "Case titles" is renamed to "Patient Information Titles".

### Bugfix:

1. Still images did not save under current Working Directory if its name was changed.
2. Calibration issues while changing the resolution causing wrong geometrical measurements.
3. Software does not close if database file is not found.
4. In some cases, patients were not loaded after selecting a database.
5. In some cases, images were not loaded properly in the image gallery.
6. Software crashed if the database path was not available. This case happened if database was saved on a removable disk and the disk was not attached to the computer.
7. Custom logotype did not save properly in report template.
8. Software hanged or crashed if Database file was not found under its specified destination folder. The issue occurred if database file was stored in an external driver, which was not plugged to the computer by mistake.
9. Changes in Case Titles (Patient Information titles) were not updated in Report if changed in Preference. Restart of software is required.