

Thank you for choosing Inspectis

This product has been tested with Windows 7/10 Professional. If you have problems running on other versions, please contact your dealer or send an email to info@inspect-is.com

Below you will find step by step instructions on how to install.

1. Install the software by running Inspectis V.4.x Setup.exe
2. Connect the included Communication Cable's circular connector to the mating connector of the camera and USB side to available USB port before starting the software. Windows will automatically detect and install the driver for the Communication Cable. If not install the drivers manually from Communication Cable Driver folder.
3. Start Inspectis. On "Select Inspectis Version" dialog box, select:
 - a. **"Inspectis ProX"** if you have ordered a **ProX** license and the software protection USB dongle. All software functions will be enabled. Insert the dongle into a free USB port of your computer. Enter your license number.
 - b. **"Inspectis Pro"** if you have ordered a **Pro** license and the software protection USB dongle. All software functions will be enabled. Insert the dongle into a free USB port of your computer. Enter your license number.
 - c. **"Inspectis Basics"** if you have ordered a **Basics** license. Image view, capture, annotation and only distance measurement on image will be enabled. Enter your license number.
 - d. 15days trial. All software functions will be enabled for 15 days. License number is not required.
4. Select your video capture adapter or any other Inspectis compatible video device and software will start monitoring live image of the digital microscope.

For more information refer to instructions manual of **Inspectis software** in PDF.

For ordering **Inspectis** software please contact your Inspectis dealer or send an email to: info@inspect-is.com.